CR60 Operating Instructions  
CR60 Bruksanvisning  
CR60 Manual de instrucciones

## Contents

Introduction	2
What to do with your new CR60	2
Applications of the new CR60	2
Part Identification	2
Electrical connection	3
To operate the CR60	3
Preparation for treatment	4
Taking the treatment	4
After the treatment	5
Sterilisation	5
Care of your CR60	6
Servicing	6
Accessories and spare parts	6
Filter changes	7
Service manuals/wiring diagrams	7
Fault finding	7
Technical specification	8
Fuse changes	8
Guarantee	8
Standards compliance	8

## Introduction

The CR60 is a high flow compressor which is designed for use with a nebulizer.

The CR60 is for medical treatment and should only be used by prescription or under the direction of a Doctor.

We are constantly improving and modifying the equipment you purchase from Profile Respiratory Systems, therefore the information contained in this publication may not exactly match the parts supplied. The alternative parts will always offer significant improvements in performance or ease of use.

## What to do with the CR60

1. Carefully unpack the CR60 (keep the box and its packaging as this may be required when returning it for servicing).
2. Complete and return the reply paid card.
3. Read the operating instructions carefully.
4. Observe our recommendations regarding accessories.

## Applications of the CR60

The CR60 compressor can be used to power most nebulizers available. The CR60's high flow is capable of nebulizing bronchodilators and similar viscosity drugs, and can deliver the more viscous antibiotic drugs as well as powering many humidifiers.

## Parts Identification


Having carefully removed the packaging you will find the new CR60 unit, and Bacteria filter (part No. 1214). We strongly advise that before using CR60 for treatment, you familiarize yourself with the accessories and equipment.

## Electrical Connection

The CR60 should be connected to a 230V ~ \*50 Hz power supply. The mains fuse located in the electrical inlet connection, is T1.0 amp. The mains plug is fitted with a 3 amp fuse.

The AC power supply cable is supplied with the CR60 and is fitted with a plug for connection via the fused electrical connector on the rear panel of the compressor. Ensure that the plug is pushed fully into the machine.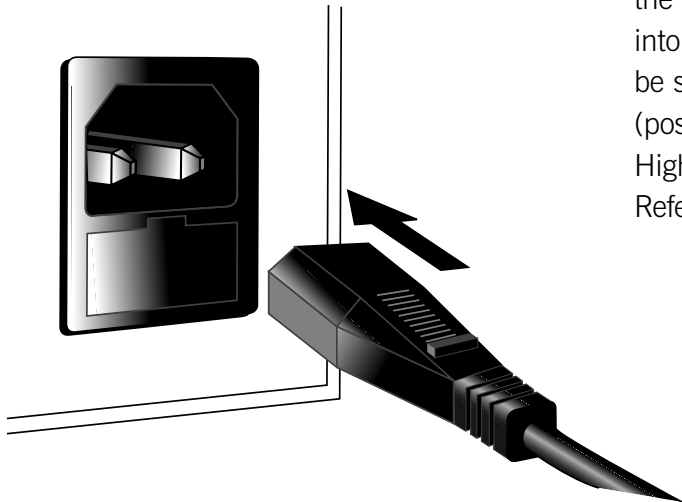

**Caution:** Before switching on the CR60 ensure that the air vents on the front and back of the machine are not obstructed, to allow cooling air to circulate.

Power cable connections are colour coded in line with British standards.

Brown – Live

Blue – Neutral

~ \* Stands for alternating current.


## To operate the CR60

First connect the machine to the mains supply via the fused electrical connector at the rear of the CR60. Ensure that the plug is pushed fully into the machine. The CR60 is now ready to be switched on at the ON/OFF switch (position I is on and position O is off) once the High Pressure Duratube is connected. Refer to “Preparation for treatment”.

Air will be drawn into the CR60 via the inlet filter on the front panel above the ON/OFF switch. (Please refer to section on “Filter changes”). Compressed air will be forced out of the compressor pressure outlet nozzle below the ON/OFF switch.

O = OFF

I = ON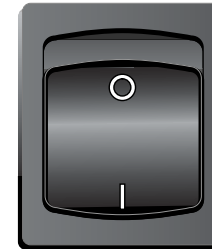

**Caution:** When the unit is switched on the air vents must not be obstructed.


## Preparing for treatment

**Caution:** It is important to understand that the drug prescribed may have to be diluted with distilled water or normal saline. If you are in any doubt about the amount to be dispensed per treatment, or the diluted factor to be used, please consult the doctor.

1. Dispense the prescribed drug directly into the top of the Sidestream nebulizer (drug chamber), or into the base by unscrewing the chamber.

**Note:** It is important that a Sidestream nebulizer is used to gain maximum performance from your CR60.

2. Connect one end of the High Pressure Duratube to the inlet on the bottom of the Sidestream nebulizer (drug chamber) and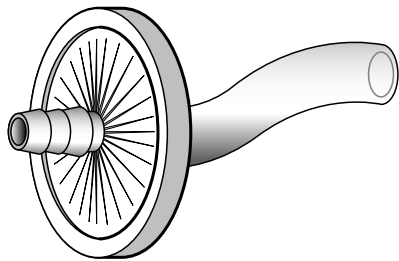

the other to the bacteria filter attached to the compressor pressure outlet. (It is essential to use a bacteria filter (part no. 1214) fitted to the air outlet of this unit).

3. Connect either a mask (part no.1100A adult or 1120A child) or mouthpiece (part no. 1601A or 1605A durable) to the Sidestream nebulizer.

The mouthpiece is provided for those who prefer this type of delivery system. Mouthpieces are generally considered to be more effective than masks. They avoid aerosol irritating the eyes and keep the face dry. Some patients may find a mask more comfortable as it can be strapped to the head with the strap provided. A child's mask is also available for the younger patient.

## Taking the treatment

Turn on the CR60 at the on/off switch and the Sidestream will produce a fine mist.

In order to get the best out of the CR60 it is necessary for the user to be instructed how to take nebulizer treatments. This is best done by your Doctor, Asthma Clinic or Physiotherapy Department at your hospital.

As a guide, it is suggested that if you are as relaxed as possible, you will find it easier to gain the maximum benefit from the treatment. Sit upright and be well supported in a chair or, if in bed, be well supported with pillows as near to upright as possible.

It is also suggested that two or three deeper breaths are interspersed with gentle lower chest breathing. Do not try and breathe quickly. Some patients find it convenient to turn the machine off during the treatment and take a short rest.

When the nebulizer begins to work intermittently and 'spits', then the treatment has finished. There is no need to try and nebulize every drop of your medication as there will always be a small amount left in the chamber.

### After the treatment

Disconnect the Sidestream nebulizer from the High Pressure Duratube by gently twisting the tubing anticlockwise. Rinse out the Sidestream and refill with a little clean water. Reconnect the High Pressure Duratube and run the Sidestream with the CR60 for a few seconds to flush out any drug that has become trapped in the nebulizer jets.


Disconnect the Sidestream from the High Pressure Duratube again gently twisting it anticlockwise. Run your CR60 for a further 15 seconds without the Sidestream to dry out any condensation in the High Pressure Duratube. Disassemble the Sidestream which comes apart in three pieces and, together with the mask or mouthpiece, thoroughly wash all the parts in warm soapy water and let them dry.

### Sterilisation

Once each week your durable Sidestream should be boiled in water with 2 drops of washing up liquid for 5 to 10 minutes.

**Caution:** When cleaning the Sidestream nebulizer (drug chamber), take great care not to damage the jet holes. Do not use a cleaning brush as this damages the centre baffle and seriously affects the performance of the Sidestream. UNDER NO CIRCUMSTANCES should you put paper clips or needles into the jet holes.

**Caution:** Do not immerse the High Pressure Duratube or the inlet filter in any solution.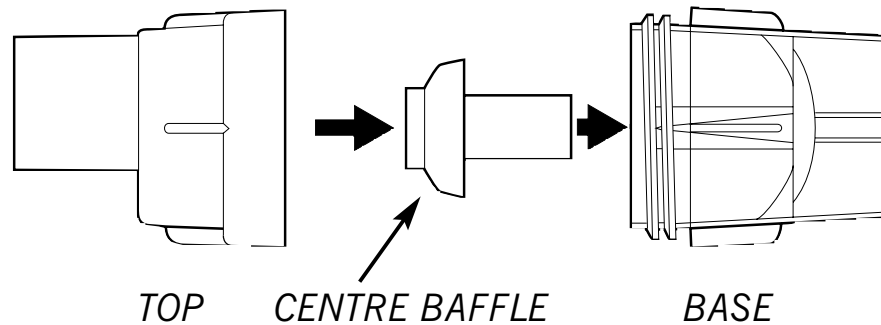

## Care of CR60

The CR60 can be cleaned occasionally by wiping the outside of the case with a damp cloth. DO NOT wipe the inlet filter. (Refer also to the section on sterilisation).

## Servicing

It is not possible to recommend a standard time between routine servicing for CR60 units as one patient may use the machine more than another. However as this medical device is to be relied upon for some years, we recommend that it is serviced by ourselves or by a qualified technician at least once a year. This will allow us to check the performance of the compressor, recalibrate it if necessary, and allow us to check the electrical integrity of the entire unit.

To assist you, we have listed possible problems (see “Fault finding”).

## Accessories and spare parts

Accessories should be replaced regularly. It is recommended that all accessories are changed at least once a year, especially the Sidestream and High Pressure Duratube if the CR60 is being used three or more times a day.

The filter should be replaced every three months (see ‘filter changes’).

If you cannot solve a problem or would like to order any spare parts, then contact our Customer Service Department and have the following information ready:

1. Unit Serial No. (The Serial No. is located at the back of the compressor).
2. Type of drug used and the frequency of treatment.

### **Profile Respiratory Systems,**

Heath Place, Bognor Regis, West Sussex, PO22 9SL, United Kingdom

Web: [www.profilehs.com](http://www.profilehs.com) Email: [info@profilehs.com](mailto:info@profilehs.com)

### **International Department:**

Telephone: +44 (0) 870 770 3500


Facsimile: +44 (0) 870 770 3501

## Filter Changes

The inlet filter is intended to filter or clean the air. In doing this the filter will get dirty and eventually blocked. The filter should be changed before it's performance and that of the CR60 deteriorates. The time span between changes varies according to the conditions the machines are used in. As a general rule as soon as the filter discolours from its normal white colour, it should be changed.

If you notice that the machine is slowing down, or it is taking longer to deliver the prescribed solution, simply remove the filter temporarily, and if the machine speeds up again, it is an indication that a new filter should now be fitted.

The filter is simply changed by unscrewing the clear filter dome, removing the old filter element and replacing with a new (Part no.2153A). Insert the new filter element

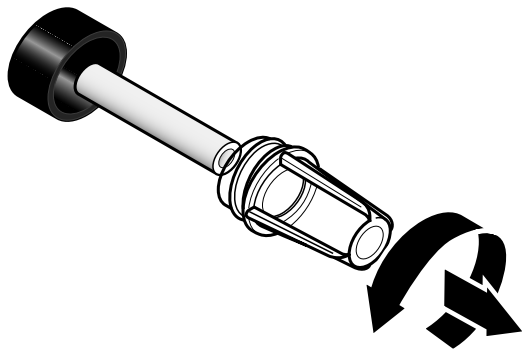into the filter holder on the machine and then replace the dome. Dispose of the old filter.

Filter changes will be required more frequently if the machine is used in a household where the occupants smoke.

The bacteria filter requires replacing once it starts to discolour (part no 1214). Dispose of the old filter.

## Fault Finding

Problem	Possible Solutions
Unit does not run	Replace fuse in plug
Switch does not light	Replace fuse in the electrical inlet connection, (disconnect unit from AC power supply before removing fuse). Push electrical connector firmly into inlet.
Switch lights, Unit runs slowly.	Check filter – Part No. 2153A
Switch lights, Unit does not run.	Return Unit for service.
Unit runs, nebulizer makes poor or no mist	Thoroughly clean Nebulizer by boiling in water with 2 drops of washing up liquid for 5 to 10 minutes. Do not use a cleaning brush. Replace nebulizer with new one.
Nebulizer bubbles but makes no mist.	Disassemble and clean the nebulizer as per above instructions. Reassemble.

If any of the faults occur and the remedies have been tried to no avail, please contact our Customer Services.

## Service manuals/wiring diagrams

Full service manuals/wiring diagrams are available to enable suitably qualified personnel to service the machines (part no 2069).

## Technical specification

Part no.	CR60 1750
Voltage	230V~*
Frequency	50Hz
Power Consumption	200VA
Fuse	1 x T1.0 amp (5 x 20mm)
Filter part no.	2020
Nominal dynamic flow*	8.5 lpm
Weight	2.8kg
Compressor wobble	'Ultra-clean'
Dimensions:	
Height	150mm
Width	155mm
Depth	225mm
Type	Class II 
	BF 
Rating	Continuous

~Stands for Alternating Current.

\*Flow through Sidestream Nebulizer Jet.


Inlet Filter-Porous Polyethylene.

CE 0120

## Fuse Changes

The mains fuse located in the electrical inlet connection, are T1.0 amp.

To gain access to the mains fuse, insert a small screwdriver into the slot on the electrical inlet connection and twist to release the door (see diagram).

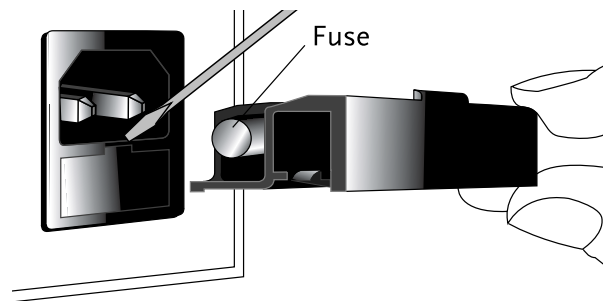The mains plug is fitted with a 3 amp fuse.

## Guarantee

The CR60 is guaranteed against defects in materials and workmanship for one year from the date of purchase.

Only Profile Respiratory Systems approved parts may be used to service this equipment. Use of non-approved parts may invalidate the Test Certificate issued for the CR60 nebulizer compressor.

This guarantee does not affect your statutory rights.

## Standard compliance

The CR60 high flow is manufactured to the requirements of IEC601-1 for class II Type BF equipment. The CR60 meets EN 60601-1-2:1993 Medical Electrical Equipment, General Requirements for Safety: Electromagnetic Compatibility.

## Transport & storage at:-

An ambient temperature of +10°C to +40°C.

A relative humidity range of 30% to 75%.

An atmospheric pressure range of 700hPa to 1060hPa.

## Symbols

Attention, consult accompanying documents.

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